

# Instructions for Autorefractor case.

The smaller black case on the right side (see picture to the right) contains the following items:

- 1) Nikon Autorefractor.
- 2) Autorefractor printer.
- 3) Autorefractor stand.
- 4) Power cable
- 5) Model eye in a bag
- 6) Fuses for Autorefractor stand.
- 7) Tester for the batteries.

**All documentation is in the smaller Optical case.**

Batteries &  
Printer paper

Battery  
Rack

Autorefractor  
in it's case



The battery rack is to the left and contains a box which includes batteries and extra rolls of printer paper.

## PLEASE FOLLOW THIS PROCEDURE WHEN YOU ARRIVE IN THE CLINIC

1. Remove the battery rack to the left of this case.
2. Carefully remove the box inside the battery rack making note as to how it fits into the rack. It will only fit one way.
3. Open the box and remove all of the batteries.
4. Using the number on the top of each battery insert it into the battery charger labeled with the same number. See insertion instructions to the right.



1) Insert the battery as shown 2) Push it down & hold 3) push to right to latch.

5. Plug the battery charger in and leave it plugged in all the time you are in the clinic.

## PLEASE FOLLOW THIS PROCEDURE WHEN YOU LEAVE THE CLINIC ON THE LAST DAY

1. Remove each battery from the charger and put it back in the box.
2. Replace the box inside the battery rack being sure to turn it the right way.
3. Put the battery rack back into the large black carrying case. The power cord is position behind the smaller autorefractor case.
4. Close the case.

**Both Cases are Open**



**Autorefractor case closed.**



**Autorefractor case fully loaded**

